

## INSTRUCTIONS FOR USE



### MEDICAL FACE MASK – WASHABLE

#### TYPE IIR cod. MC.001.005

**Special splash resistant mask, protects against blood and body fluids  
Suitable for use in the operating room or other similar conditions**

#### *Medical Device T0206 according to CND nomenclature*

Directive 93/42/EEC – Legislative Decree 46/97

### GENERAL DESCRIPTION

The medical mask MC.001.005 is designed to minimize the transfer of pathogenic microorganisms and particles from the healthcare professional to the patient. It is also able to reduce the possibility of respiratory infection if worn by a sick person.

The medical mask MC.001.005 has been designed to ensure a high level of protection combined with good comfort, based on the international standard UNI EN 14683 (version 2019).

It is suitable for use in the operating room or other similar conditions, offering special resistance and protection against splashes of blood and body fluids. In particular situations of epidemics or pandemics it can also be used by other people to reduce interpersonal contagion.

### TECHNICAL FEATURES

The mask MC.001.005 is WASHABLE and REUSABLE. It does not contain latex. It is not sterile. It is made of non-woven fabric (100% polypropylene) that is water-repellent and breathable.

### HOW TO WEAR THE MASK

- *Handle the mask with perfectly clean hands*
- *Insert your fingers between the elastic ear bands*
- *Place the mask on the face making sure to cover the nose and mouth and the tip of the chin*
- *Place the elastic bands around the ears*
- *Pull the surgical mask from top and bottom to completely unfold the folds*
- *During use, do not touch it with your hands or with potentially contaminated material*
- *Replace the mask if contaminated or when it becomes wet with breathing*
- *Dispose of the used mask according to current regulations (as biohazard sanitary waste if it is contaminated)*

### CONTRAINDICATIONS

Do not use in case of obvious damage or defects (holes, tears, loose bands, etc.).

### STORAGE

Store at room temperature, without exposing to humidity or direct heat.



## MAINTENANCE

The masks are reusable and can be washed up to 30 times before disposing of them completely.

The mask must be washed according to the following instructions:

1. Immerse the mask in a container containing a chlorine solution with a dilution of 0.1%
2. leave it immersed for 10 minutes
3. remove it and immerse it in a second container containing clean water and rinse thoroughly
4. remove it, squeeze it lightly and immerse it in a container containing clean water (if you use the same container as in point 3, replace the water already used for the first rinse)
5. leave to dry completely without using a hairdryer or dryer


**Note:** Among the products based on active chlorine there is the common bleach which on the market has a 5-10% chlorine content.

### How to arrive at a dilution of 0.1% in active chlorine:


read the product label carefully, checking the percentage of chlorine and then dilute it in water to the right extent.

Here are some examples:

If we use a product with **5%** chlorine to obtain the right percentage of **0.1%** of active chlorine it must be diluted in this way:

 100 ml of product (5%) in 4900 milliliters of water

or

 50 ml of product (5%) in 2450 milliliters of water

## DISPOSAL

If contaminated, dispose of after use as infectious biological waste.