

# SCALABROS SRL

Via Caduti di Sabbiuno 83/C,  
Loc. Bazzano  
40053 Valsamoggia (BO)  
Tel: 051- 832587– E-mail: info@scalabros.it  
C.F. 02853661201



## INSTRUCTIONS FOR USE



### ELASTIC CAP model CF.001.001

*Medical device T0207 according to CND nomenclature*

Directive 93/42/EEC – Legislative Decree. 46/97

### GENERAL DESCRIPTION

The cap CF.001.001 offers a basic protection to the operator and patient for the transfer of infectious agents, particularly it minimizes the transfer of microorganisms, body fluids and particles, including those released by the scalp, between patients and operators.

### TECHNICAL FEATURES

The DISPOSABLE cap is made of hydrophobic and breathable non-woven fabric (100% polypropylene). It is latex free. The elastic band is in synthetic rubber.

### HOW TO WEAR THE CAP

- *Before entering the working area, take the cap out of the package*
- *Handle the cap with clean and dry hands, then put it on the head*
- *Lay out the cap in order to pull your hair back and leave free the field of view*

### CONTRAINDICATIONS

Do not use in case of obvious damage or defects (holes, tears, etc).

### STORAGE

Store at room temperature, without exposing to humidity or direct heat.

### DISPOSAL

If contaminated, dispose after use as infectious biological waste.