

SCALABROS SRL

Via Caduti di Sabbiuno 83/C,
Loc. Bazzano
40053 Valsamoggia (BO)
Tel: 051- 832587– E-mail: info@scalabros.it
C.F. 02853661201



INSTRUCTIONS FOR USE



DISPOSABLE SURGICAL GOWN model CC.001.001

Medical Device T0204 according to CND nomenclature

Directive 93/42/EEC – Legislative Decree 46/97

GENERAL DESCRIPTION – WARNINGS

The surgical gown CC.001.001 offers a basic protection to the operator and patient for the transfer of infectious agents, in accordance with the standard UNI EN 13795-1:2019.

The guaranteed performances refer to the Prospectus 1 of the standard, which provides that the medical gown is not suitable for invasive surgical procedures, where it's expected a long exposure to liquids, mechanical stress or for extended surgeries.

TECHNICAL FEATURES

The DISPOSABLE surgical gown is made of hydrophobic and breathable non-woven fabric (100% polypropylene). It is latex free. Cuffs are made of stretch cotton.

Before the use, it can be sterilised with validated methods.

HOW TO WEAR THE MEDICAL GOWN

- *Surgical gowns must be worn with the help of a second person*
- *Take the surgical gown from the package with perfectly clean and dry hands or with clean gloves*
- *(If it requires a sterilization before the use, the customer can perform it following validated procedures)*
- *Lay out the surgical gown on a clean area*
- *Slide your arms through the sleeves of the gown*
- *Slip both hands and forearms in the sleeves, pulling your arms all the way through so that the gown is now hanging off your shoulders and your hands out of the cuffs*
- *Ask to your assistant to tie the laces behind your back.*

CONTRAINDICATIONS

Do not use in case of obvious damage or defects (holes, tears, etc)

STORAGE

Store at room temperature, without exposing to humidity or direct heat.

DISPOSAL

If contaminated, dispose of after use as infectious biological waste.